

# LÜNEBURG WEIMER

## MATERIAL SAFETY DATA SHEET (MSDS)

### INSTRUCTIONS FOR DENTAL COTTON ROLLS

#### I. USAGE:

Used for absorbing saliva, blood while dental treatment, ensure dry working place of the suffering teeth

#### II. TECHNOLOGY DATA:

Tech Specification	Diameter range: 0.8cm; 1.0cm; 1.2cm; 1.5cm +/- 0.1 – 0.15cm, length range: 3.8cm +/-0.3cm
Material	100% cotton

#### III. APPLICATION:

1. Used by nurse or doctor or under the instruction of them
2. Choose suitable size of disposable dental cotton rolls, according to the position of suffers' wound and position.
3. Open the package, take out the small bundle bag, take one piece and ready for use

#### IV. PACKAGE:

2000pcs / box, 1000pcs / bag ,600 pcs/bag. Special package details are marked on the package

#### V. STORAGE:

Store in the well-ventilated rooms, temperature control  $\leq 40^{\circ}\text{C}$ , moisture  $\leq 80^{\circ}$

#### VI. SHELF LIFE:

Expiration date of the products is 3 years, do not use if out of expiration date

#### VII. PRECAUTIONS:

1. Disposable use, not for repetition used
2. Do not use if out of expiration date
3. Do not use if any damage of product

Manufactured by [Lüneburg Weimer](#)